

NEEDLE RETRACTABLE SAFETY SYRINGE

+ Feature & Benefits




- Fingers stay behind the needle at all times;
- Non-exposure of needle after manual retraction;
- Requires minimum training;
- No add-on pieces allows for low angle injections;
- Retract speed under control - no splatter after activation;
- Competitive price advantage over other safety technologies.

Syringe size							
0.05ml	0.1ml	0.3ml	0.5ml	1ml	2-3ml	5ml	10ml
Insulin Syringe size (U-100 or U-40)							
0.3ml		0.5ml		1ml			
Needle gauge							
18G	19G	20G	21G	22G	23G		
24G	25G	26G	27G	29G	30G		



Also available with other needle size on request

+ How to use

<p>Be sure needle fix tightly and draw up medicine using standard procedure</p> 	<p>Give injection using aseptic techniques, fully depress the plunger to lock the needle</p> 
<p>Pull plunger back until needle fully be retracted into the barrel.</p> 	<p>Snap off the plunger, the used needle will be encased inside the barrel</p> 